


mm	60	76	91	101	111	121	126	136	141	151	161	173	186	198
	75	90	100	110	120	125	135	141	150	160	172	185	197	207
Size	<b>00</b>	<b>0</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>4.5</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>
in	2.3	3.0	3.6	4.0	4.4	4.8	5.0	5.4	5.6	6.0	6.3	6.8	7.3	7.8
	2.9	3.5	3.9	4.3	4.7	4.9	5.3	5.5	5.9	6.2	6.7	7.2	7.7	8.1



### Step 1:

Clean the hoof and measure either the length or width of the hoof as shown in the diagram to the right.

If due to an underlying condition there are significant differences in the length and the width of the horse's hoof, always take the larger dimension. This also applies if the measurements of both front or hind feet differ between left and right, the bigger mould will fit both. Consider, that the mesh will add approx. 2 mm depending on the amount of layers used in the applicatoin process.

### Step 2:

Check the FormaHoof sizing chart and select a size range which fits with your measurement. Do not include bulbs of the heel in the measurement.

### Step 3:

You are ready to order your Formahoof mould.

If you are presented with a special case or injury, or are just unsure of which size to purchase, you can reach one of our FormaHoof experts via email to receive guidance. We strongly recommend that you include photos of your horse's hooves and the measurements in your emails, so our experts can assist you faster.

